GOJO SMARTLINK™

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HH Compliance

Filters

Shifts	Protocol Moments	Job Roles
<input type="checkbox"/> 1ST	<input type="checkbox"/> In Room	<input checked="" type="checkbox"/> Physician
<input type="checkbox"/> 2ND	<input type="checkbox"/> Out of Room	<input type="checkbox"/> Attending
<input checked="" type="checkbox"/> 3RD		<input type="checkbox"/> Head Nurse
		<input type="checkbox"/> Nurse
		<input type="checkbox"/> House Staff

NHSN Codes
You don't have any NHSN codes set up at this time.

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HH Compliance - Full Facility Report
5/2/2016 12:00 AM - 5/9/2016 3:39 PM EST

Export

Date	Total Successful Outcomes	Opportunities	% Comp
3/6/2016	1	1	
3/13/2016	1	2	
3/20/2016	1	1	
3/27/2016	4	5	
4/3/2016	2	2	
4/10/2016	0	0	
4/17/2016	0	0	
4/24/2016	0	0	
5/1/2016	0	0	
5/8/2016	0	0	
5/15/2016	1	1	
5/22/2016	1	2	
5/29/2016	4	9	44%
6/5/2016	0	0	

Demo Hospital - In / Out
Hand Hygiene Observation Performance History
Full Facility Report Date Range: 11/1/2015 12:00 AM - 5/31/2016 10:56 AM EST Generated: 5/31/2016 10:56 AM EST

24 COMPLIANT 33 TOTAL 72.7% PERFORMANCE

Overall Performance Summary

Recorded Time	Unit	Floor	Site	Observer	Department	Session ID	Session Observer	Job	Moment	Outcome	Feedback Provided	Notes	Device
6/3/2016 16:14	Emergency Department	Floor 1	Demo Hospital - Five Moments	IPC		12928	Demo User	Nurse	Before Contact	Wash	Not Selected		iPhone
6/3/2016 16:14	Emergency Department	Floor 1	Demo Hospital - Five Moments	IPC		12928	Demo User	Nurse	After Exposure	Rub	Not Selected		iPhone
6/3/2016 16:14	Emergency Department	Floor 1	Demo Hospital - Five Moments	IPC		12928	Demo User	Sister	After Environment	Rub	Not Selected		iPhone
6/3/2016 16:14	Emergency Department	Floor 1	Demo Hospital - Five Moments	IPC		12928	Demo User	Other	Before Procedure	None	Yes	Poor Technique;	iPhone

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HH Activity History - Full Facility Report Export

Activity by Unit

Activity by Unit
 Activity by Job Role

UNIT	QUARTERLY TREND	MAY		Q2 TO DATE		Q1 2016		Q4 2015		Q3 2015		YEAR TO DATE	
		%	#	%	#	%	#	%	#	%	#	%	#
2 East	↓	0.0%	0	0.0%	0	0.0%	0	100.0%	2	50.0%	2	0.0%	0
2 West	↑	0.0%	0	0.0%	0	100.0%	1	60.0%	5	0.0%	0	100.0%	1
Clinic	↓	0.0%	0	50.0%	2	0.0%	0	71.4%	7	0.0%	0	50.0%	2
Emergency Department	↓	0.0%	0	100.0%	4	0.0%	0	100.0%	1	0.0%	0	100.0%	4
MICU	↓	0.0%	0	0.0%	0	0.0%	0	50.0%	2	0.0%	0	0.0%	0
TOTALS		0.0%	0	83.3%	6	100.0%	1	70.6%	17	50.0%	2	85.7%	7

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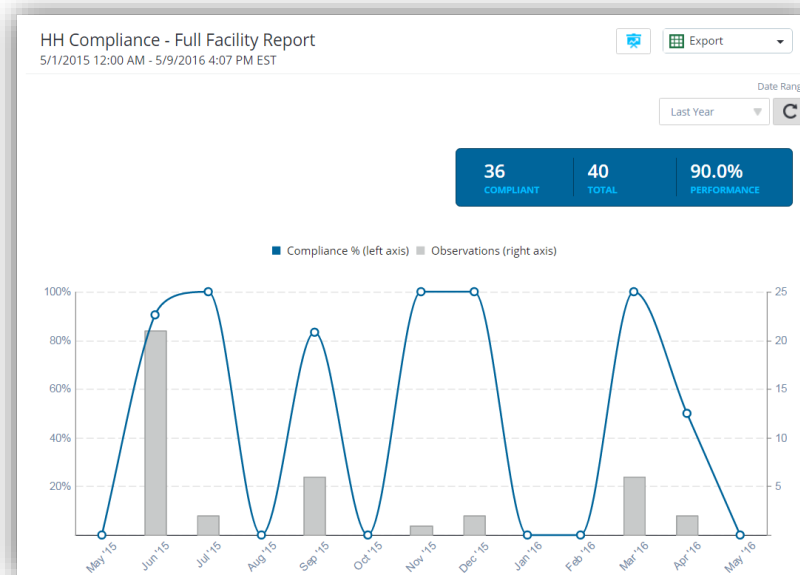
Overall Performance Summary


Compliant Observations		Total Observations		Hand Hygiene Performance	
This Week	25	This Week	30	This Week	83.3%
This Month	91	This Month	123	This Month	74.0%
This Year	117	This Year	157	This Year	74.5%
Total	117	Total	157	Total	74.5%

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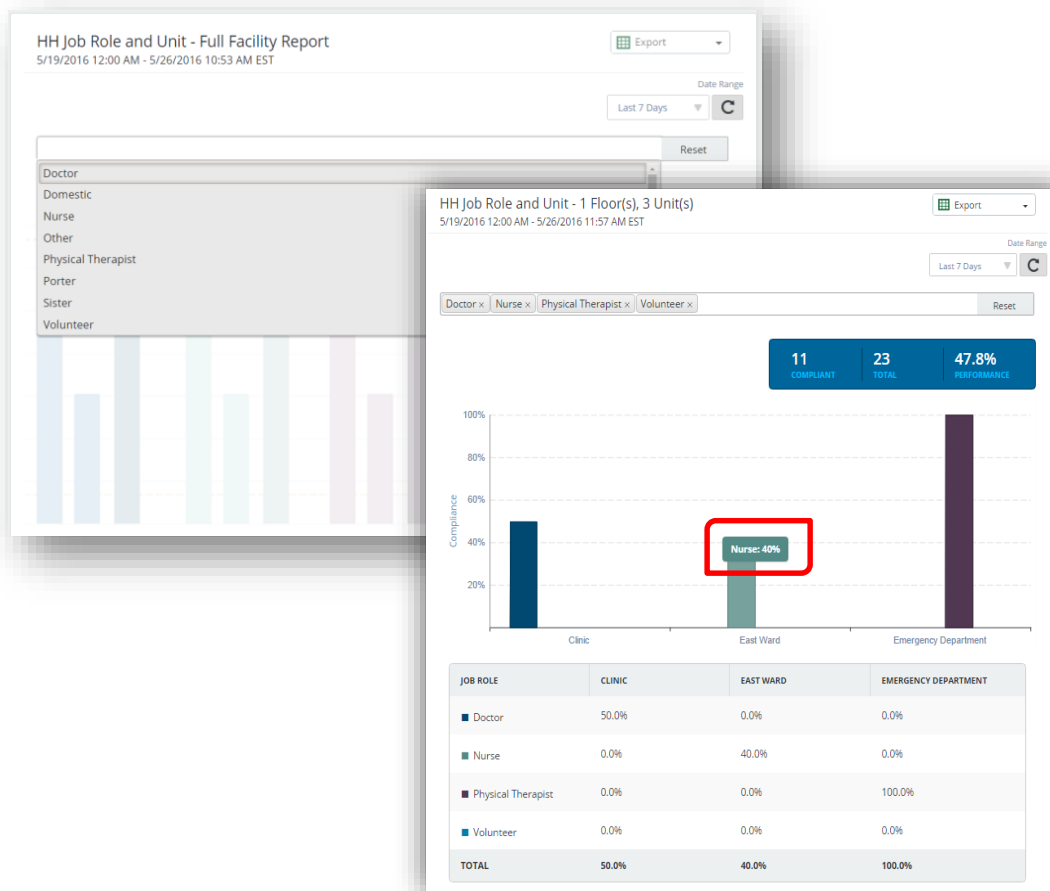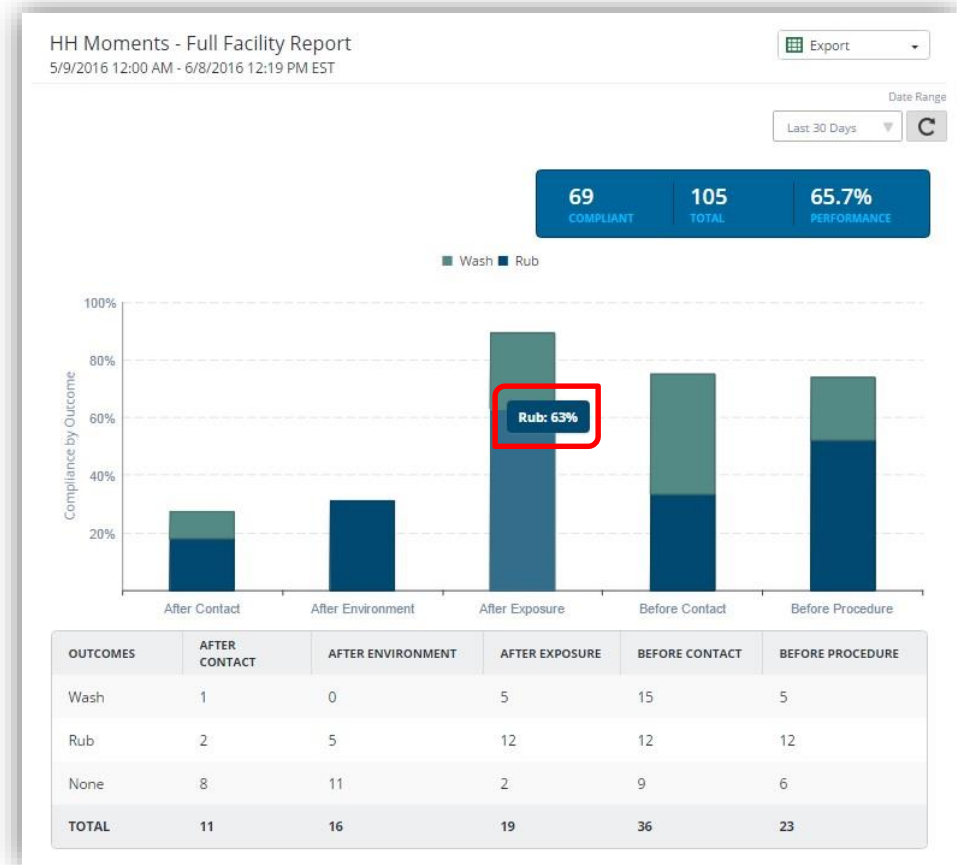
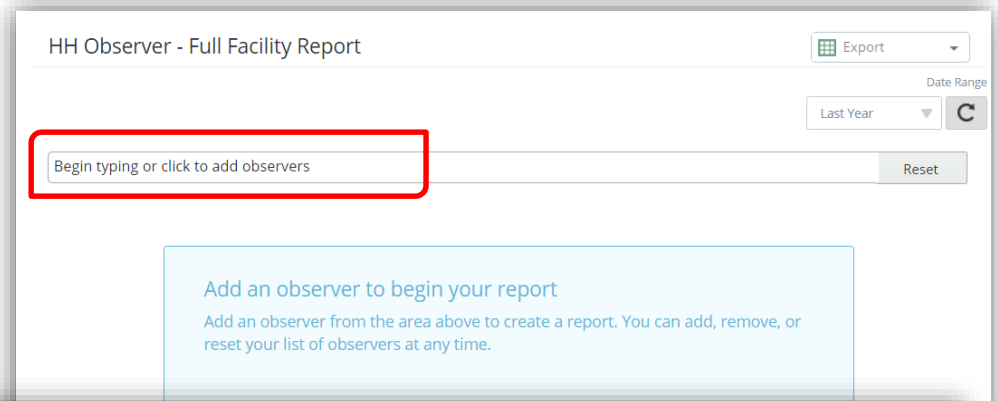
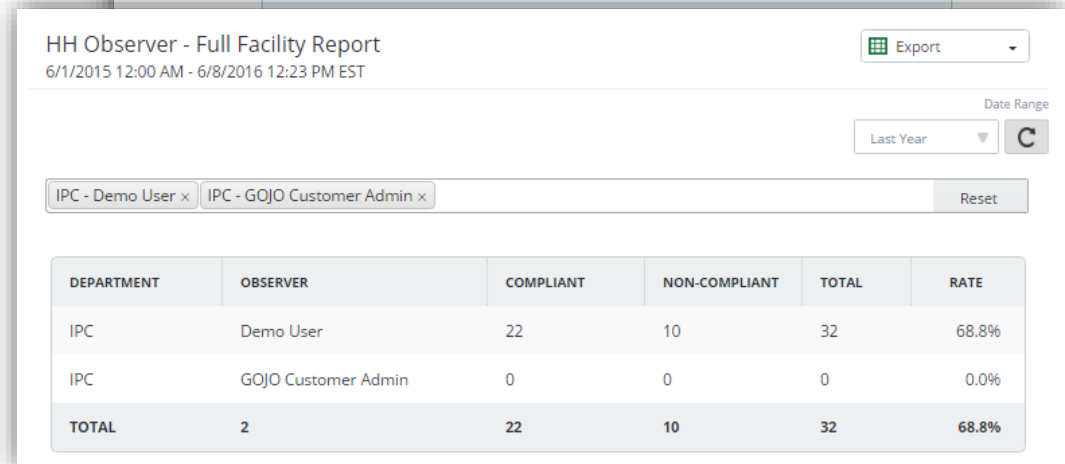
Begin typing or click to add observers

Reset

Add an observer to begin your report
Add an observer from the area above to create a report. You can add, remove, or reset your list of observers at any time.

HH Observer - Full Facility Report

6/1/2015 12:00 AM - 6/8/2016 12:23 PM EST

Export

Date Range: Last Year

IPC - Demo User x IPC - GOJO Customer Admin x

Reset


DEPARTMENT	OBSERVER	COMPLIANT	NON-COMPLIANT	TOTAL	RATE
IPC	Demo User	22	10	32	68.8%
IPC	GOJO Customer Admin	0	0	0	0.0%
TOTAL	2	22	10	32	68.8%

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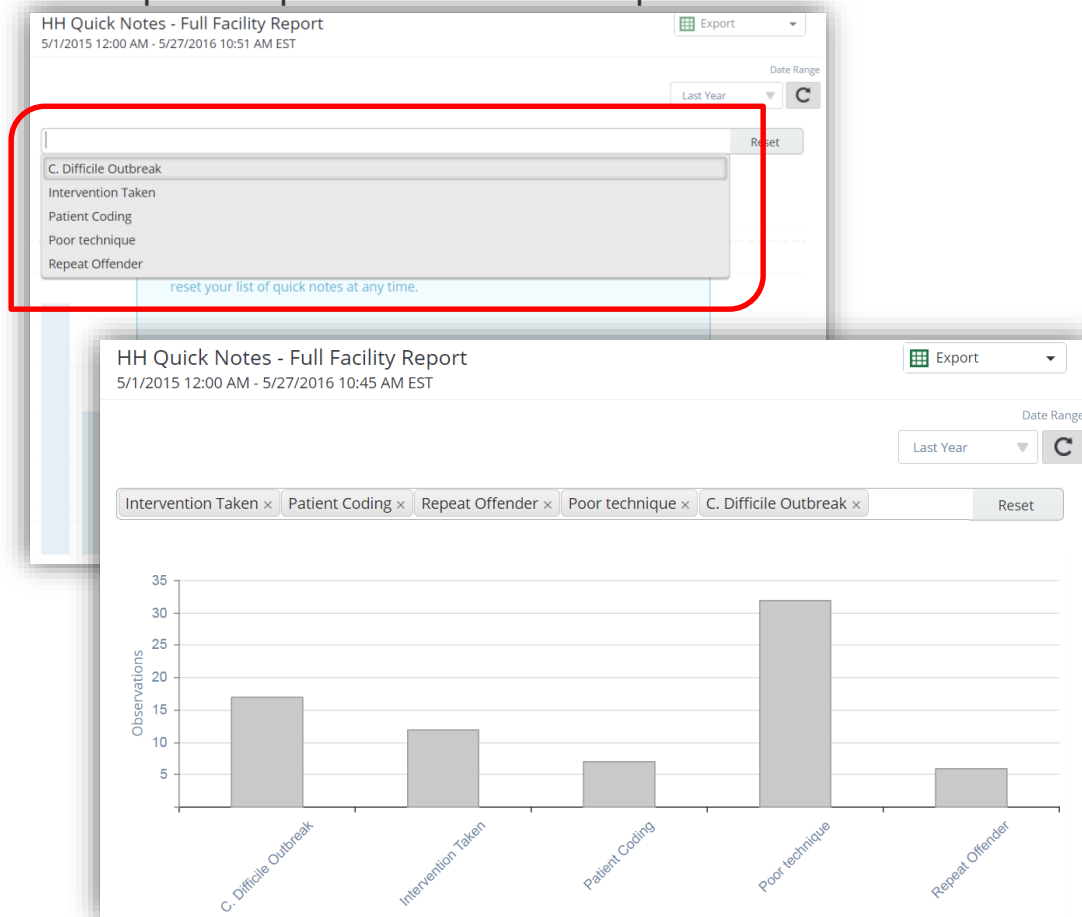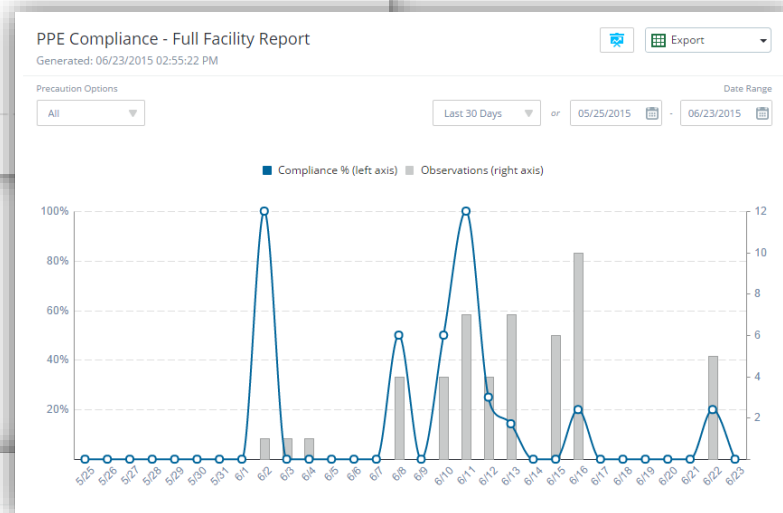
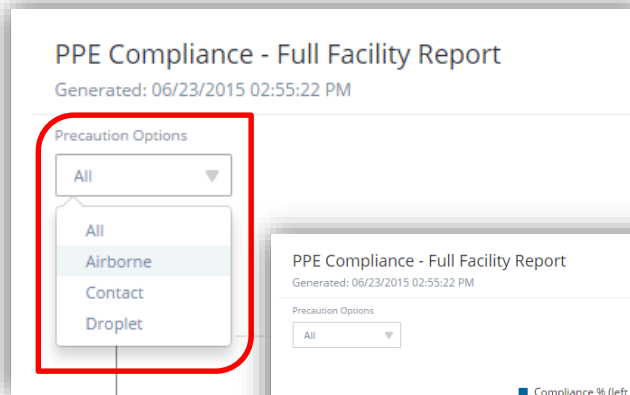
Overall Performance Summary		
Compliant Observations	Total Observations	PPE Performance
This Week 1	This Week 5	This Week 20.0%
This Month 17	This Month 50	This Month 34.0%
This Year 24	This Year 66	This Year 36.4%
Total 24	Total 66	Total 36.4%

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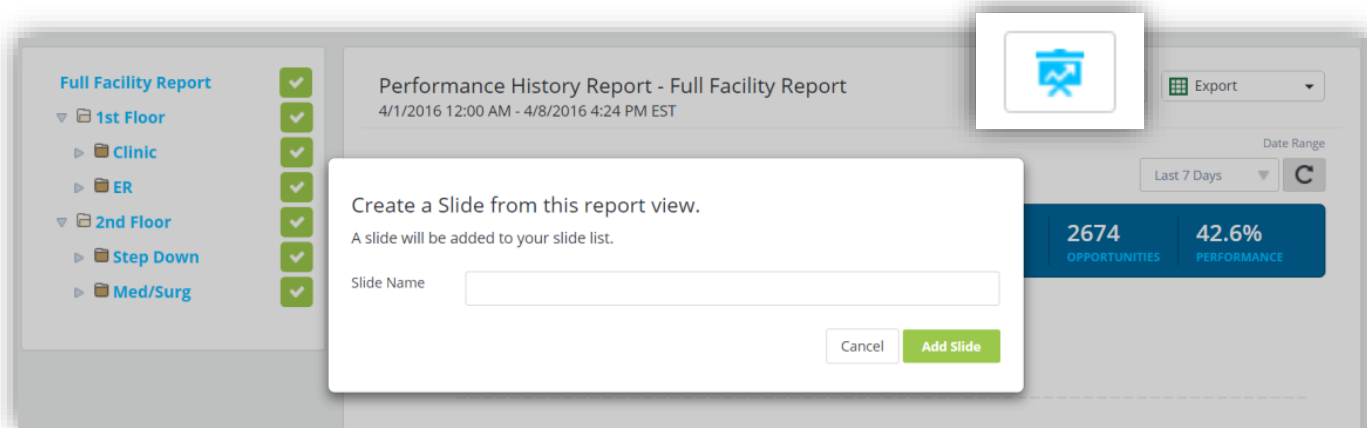
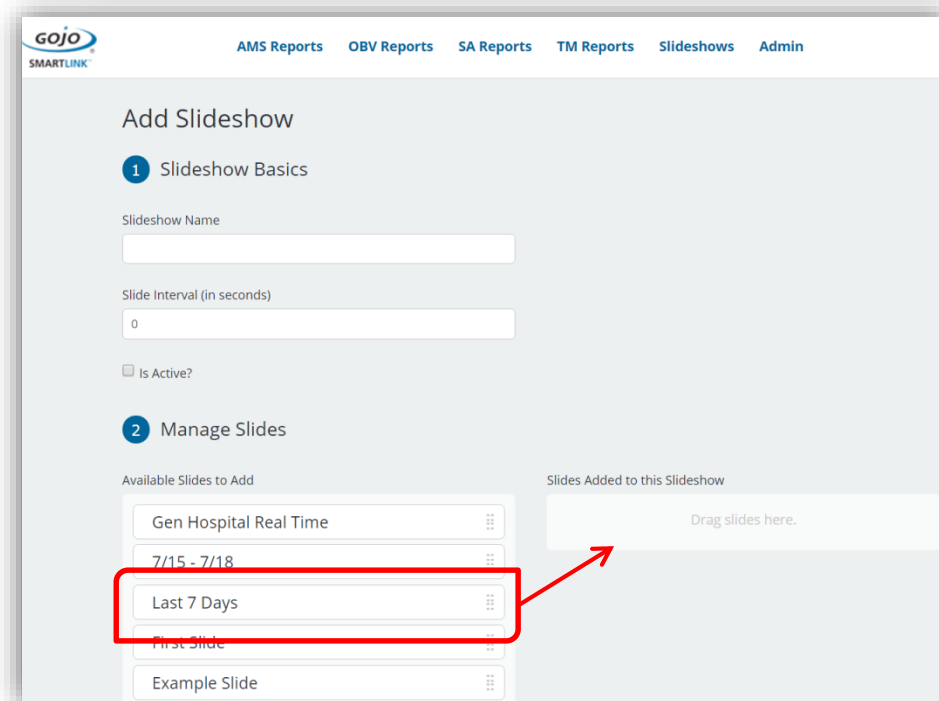
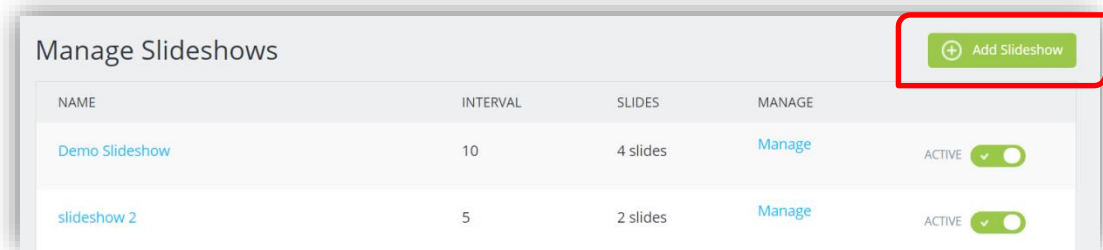
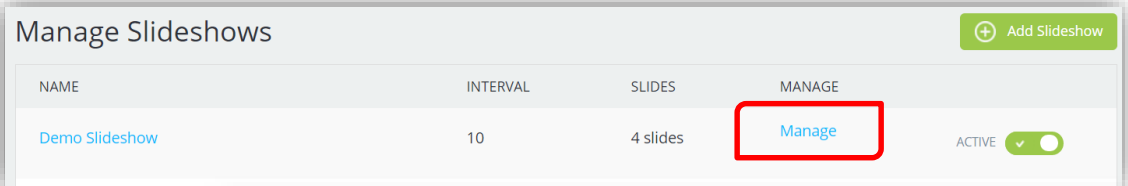
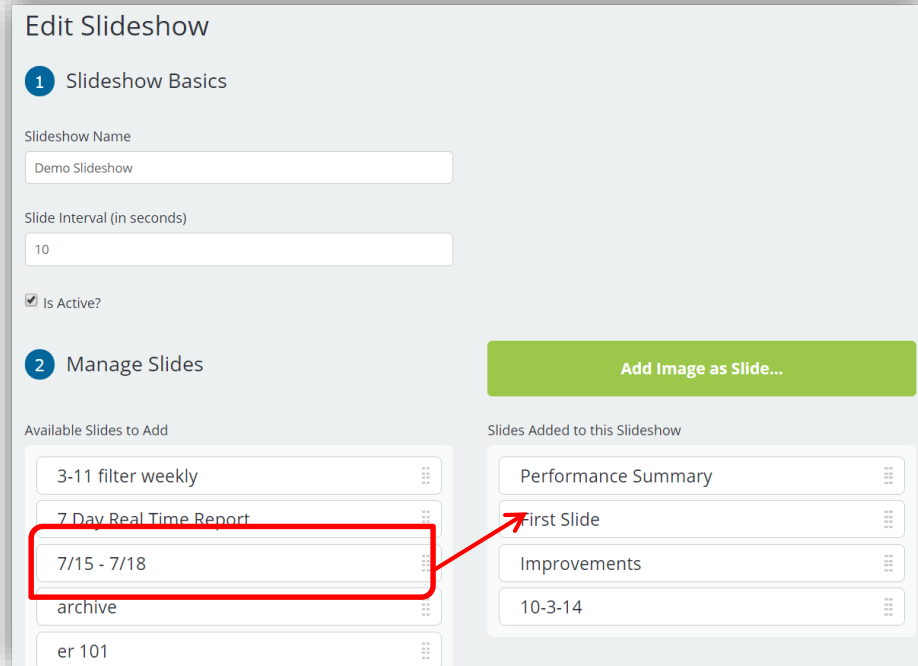
Slides Added to this Slideshow

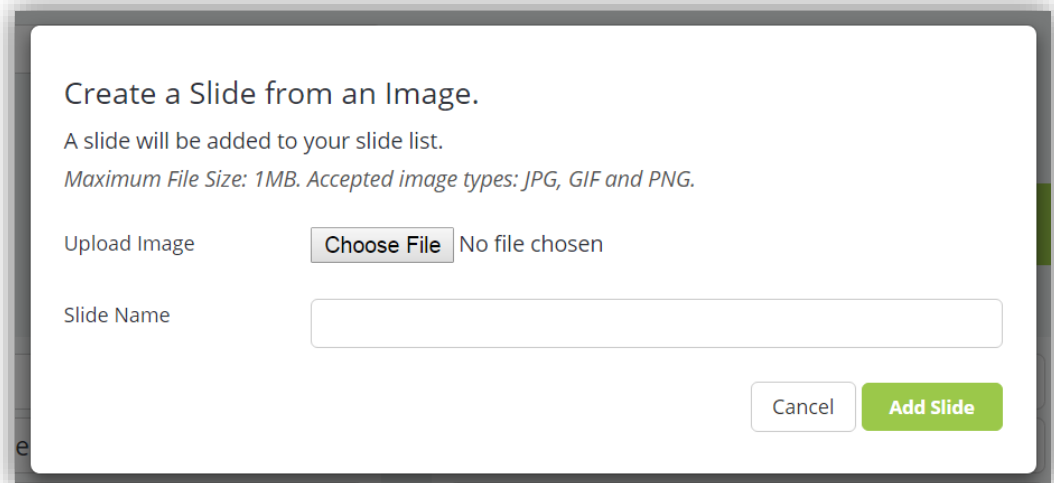
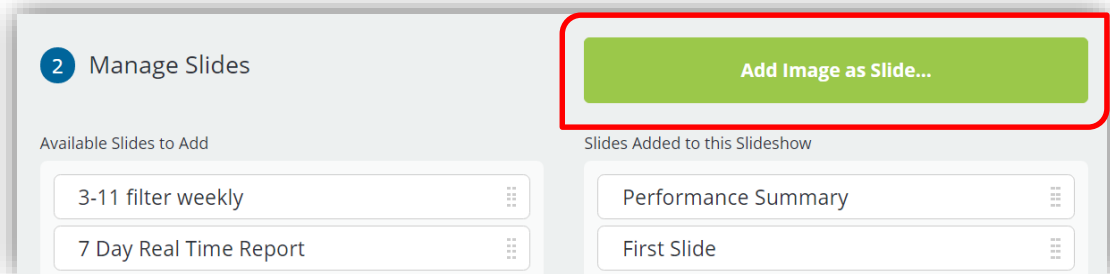
- Performance Summary
- First Slide
- Improvements
- 10-3-14

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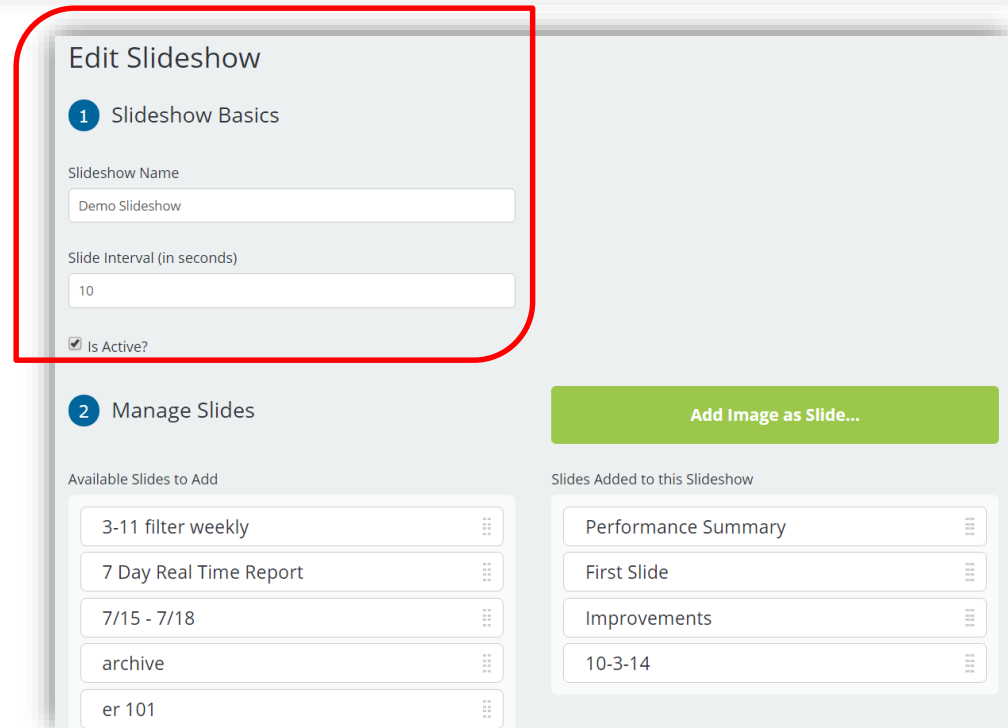
Slides Added to this Slideshow:

 - Performance Summary
 - First Slide
 - Improvements
 - 10-3-14

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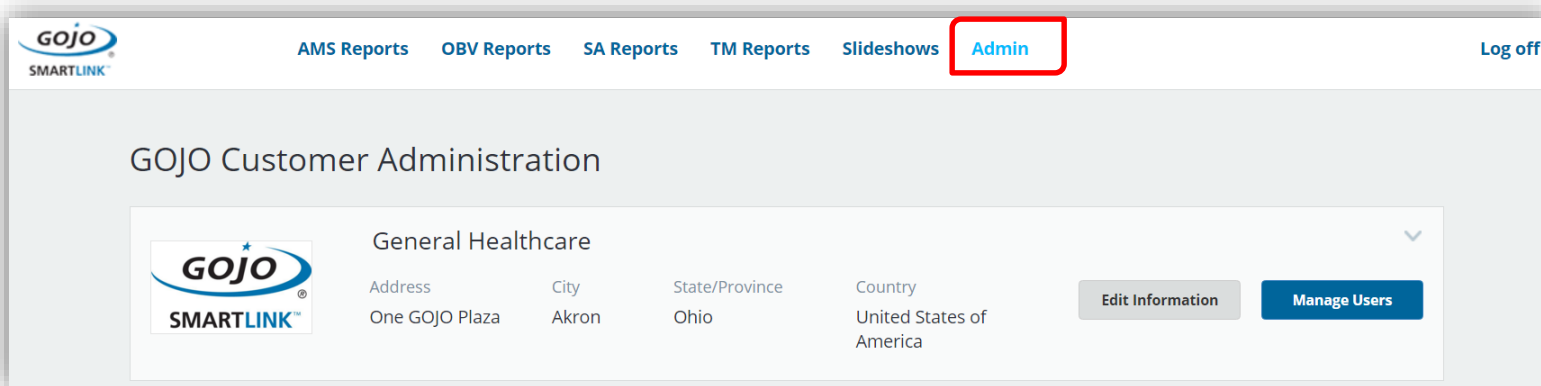
GOJO SMARTLINK

AMS Reports OBV Reports SA Reports TM Reports Slideshows **Admin** Log off

GOJO Customer Administration

General Healthcare

GOJO SMARTLINK	Address	City	State/Province	Country	Edit Information	Manage Users
	One GOJO Plaza	Akron	Ohio	United States of America		

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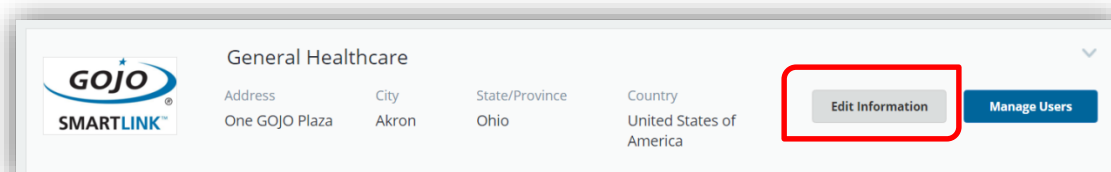
The screenshot shows a customer profile card for 'General Healthcare'. It includes the GOJO SMARTLINK logo, the customer name, and a table of address details:

Address	City	State/Province	Country
One GOJO Plaza	Akron	Ohio	United States of America

At the bottom right of the card, there are two buttons: 'Edit Information' (highlighted with a red box) and 'Manage Users'.

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
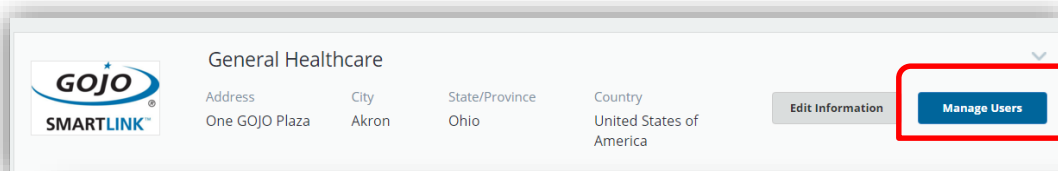
Address	City	State/Province	Country
One GOJO Plaza	Akron	Ohio	United States of America

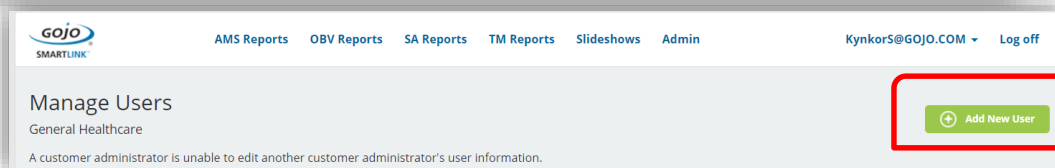[Edit Information](#) [Manage Users](#)


AMS Reports OBV Reports SA Reports TM Reports Slideshows Admin KynkorS@GOJO.COM Log off

Manage Users
General Healthcare

[Add New User](#)

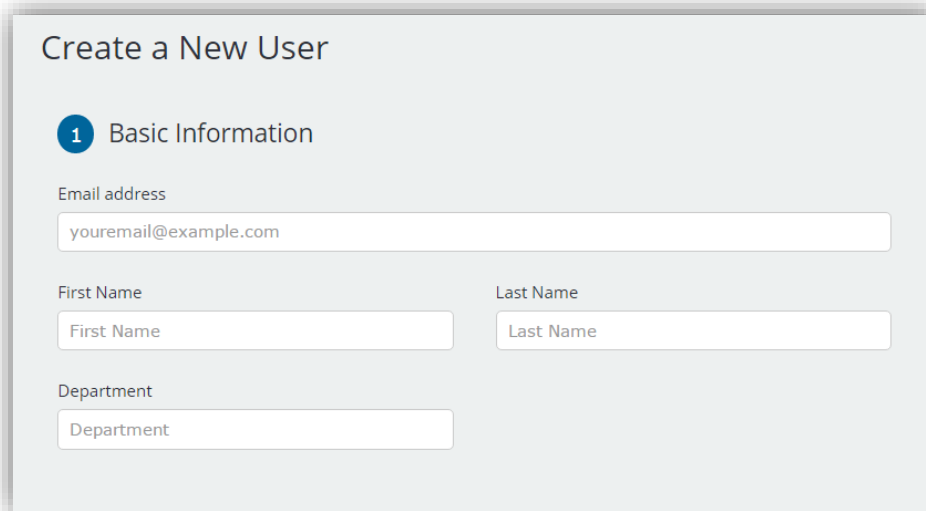A customer administrator is unable to edit another customer administrator's user information.

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2 Permission Roles ⓘ

Facility: All Facilities
Select the user's facility.

Permission Role: Data Analyst
Select the permissions for this user.

+ Add another role for this user
Add another Facility and Permission Role if this employee also works at another facility

Cancel Add New User

2 Permission Roles ⓘ

Facility: All Facilities
Select the user's facility.

Permission Role: Data Analyst
Select the permissions for this user.

+ Add another role for this user
Add another Facility and Permission Role if this employee also works at another facility

2 Permission Roles ⓘ

Facility: All Facilities
Select the user's facility.

Permission Role: Data Analyst
Select the permissions for this user.

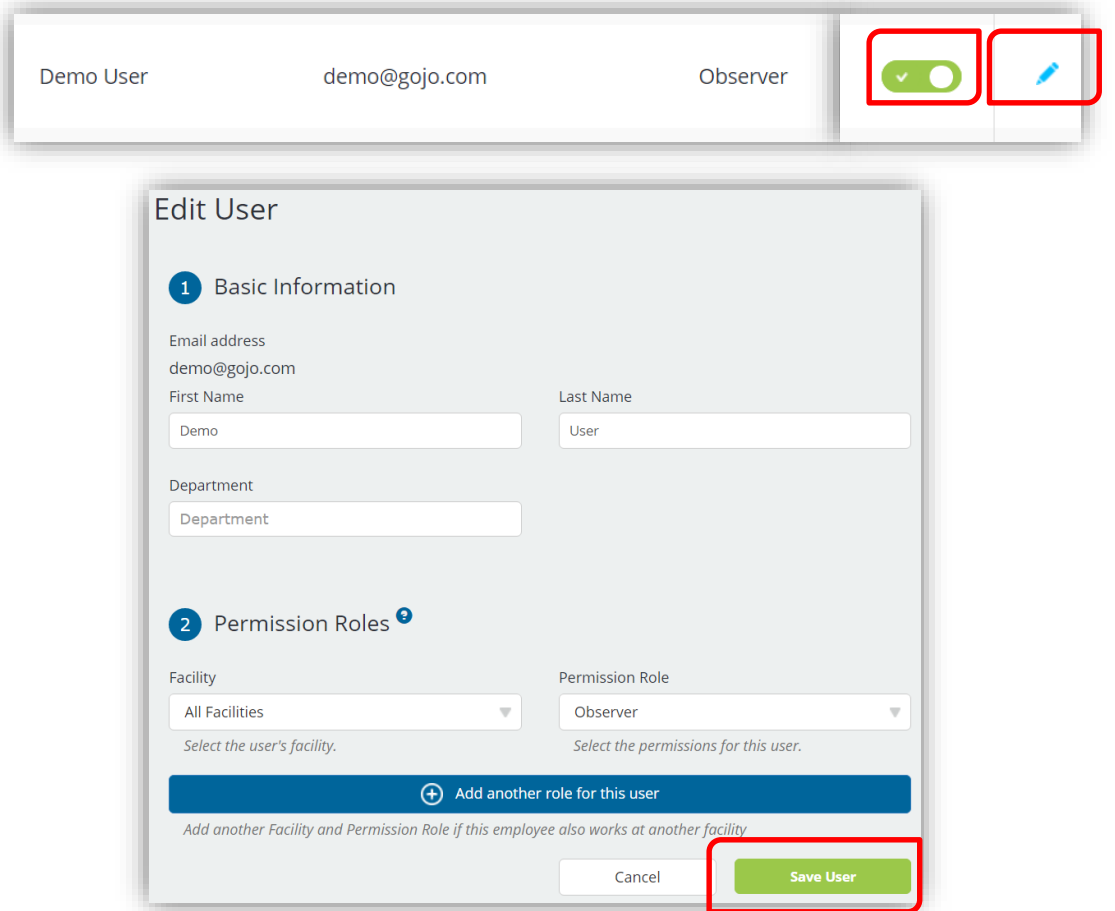
+ Add another role for this user
Add another Facility and Permission Role if this employee also works at another facility

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The screenshot displays the user management interface. At the top, a user card for 'Demo User' is shown with email 'demo@gojo.com' and role 'Observer'. To the right of the card are two icons: a green checkmark in a circle (highlighted with a red box) and a blue pencil icon (also highlighted with a red box). Below the card is the 'Edit User' form, which is divided into two sections: '1 Basic Information' and '2 Permission Roles'. The 'Basic Information' section includes fields for 'Email address' (demo@gojo.com), 'First Name' (Demo), 'Last Name' (User), and 'Department' (Department). The 'Permission Roles' section includes a 'Facility' dropdown (All Facilities) and a 'Permission Role' dropdown (Observer). Below these dropdowns are instructions: 'Select the user's facility.' and 'Select the permissions for this user.'. At the bottom of the form, there is a blue button with a plus icon and the text 'Add another role for this user', and a note: 'Add another Facility and Permission Role if this employee also works at another facility'. At the very bottom right of the form, there are two buttons: 'Cancel' and 'Save User' (highlighted with a red box).

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General Healthcare

Address	City	State/Province	Country
One GOJO Plaza	Akron	Ohio	United States of America

Edit Information Manage Users

GOJO Customer Administration

General Healthcare

Address	City	State/Province	Country
One GOJO Plaza	Akron	Ohio	United States of America

Edit Information Manage Users

5 Facilities 65 Devices

Add Facility

FACILITY	DEVICES	ACTIONS
Demo Hospital - Five Moments	0	[Edit] [Manage Users] [Add] [Heart]
Demo Hospital - Four Moments	0	[Edit] [Manage Users] [Add] [Heart]
Demo Hospital - In / Out	1	[Edit] [Manage Users] [Add] [Heart]
General Hospital (Demo)	64	[Edit] [Manage Users] [Add] [Heart]
Hospital General Regional No 251	0	[Edit] [Manage Users] [Add] [Heart]

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Manage Job Roles

Add or change job roles for your facility. Changes to the job roles will be automatically updated for SmartLink™ mobile users during their next login.

Add a New Job Role

The job role allows for personalized descriptions of jobs that are specific to your facility. The job type is a standardized list of hospital roles. Enter all the possible job roles for which an observer would observe hand hygiene compliance. Select the job type that most closely matches each job role.

Select Job Type Add Job Role

For example: ER Nurse, Surgical Assistant, Radiation Doctor, etc.

Job Roles for General Hospital (Demo)	
Attending	Edit Job Role Name
EVS	Edit Job Role Name
Head Nurse	Edit Job Role Name
House Staff	Edit Job Role Name
Nurse	Edit Job Role Name
Physician	Edit Job Role Name

Select Job Type

- Administration
- Chaplain/Clergy/Pastoral Care
- Clinical Nurse Consultant
- Clinical Technician
- Clinical Nurse Specialist
- Dialysis Technician
- Dietitian
- Domestic
- Environmental Services Worker
- Facility/Maintenance Worker
- Fellows
- Healthcare Assistant
- Lactation Consultant


Job Roles for Demo Hospital - Five Moments

Doctor	Edit Job Role Name	Archive
Domestic	Edit Job Role Name	Archive
Nurse	Edit Job Role Name	Archive
Other	Edit Job Role Name	Archive
Physical Therapist	Edit Job Role Name	Archive

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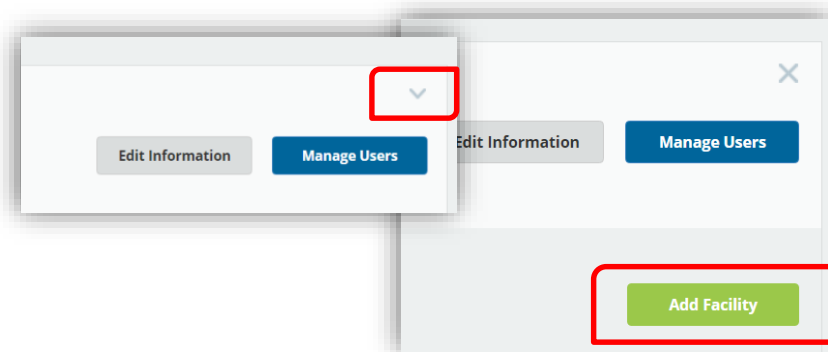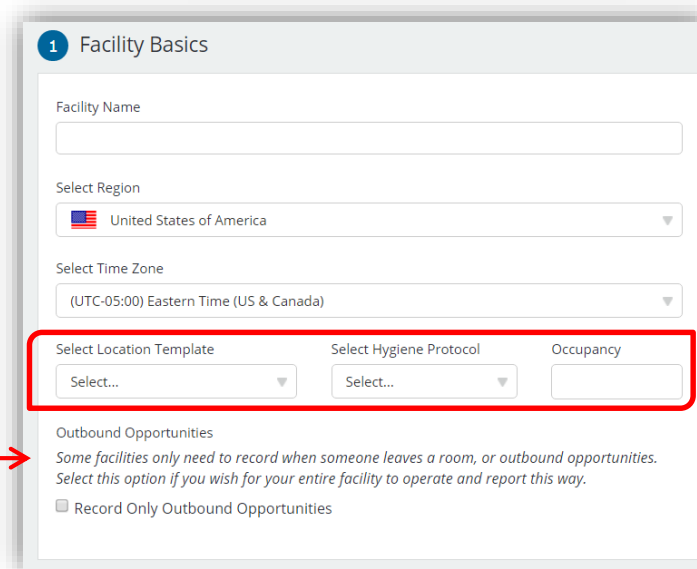
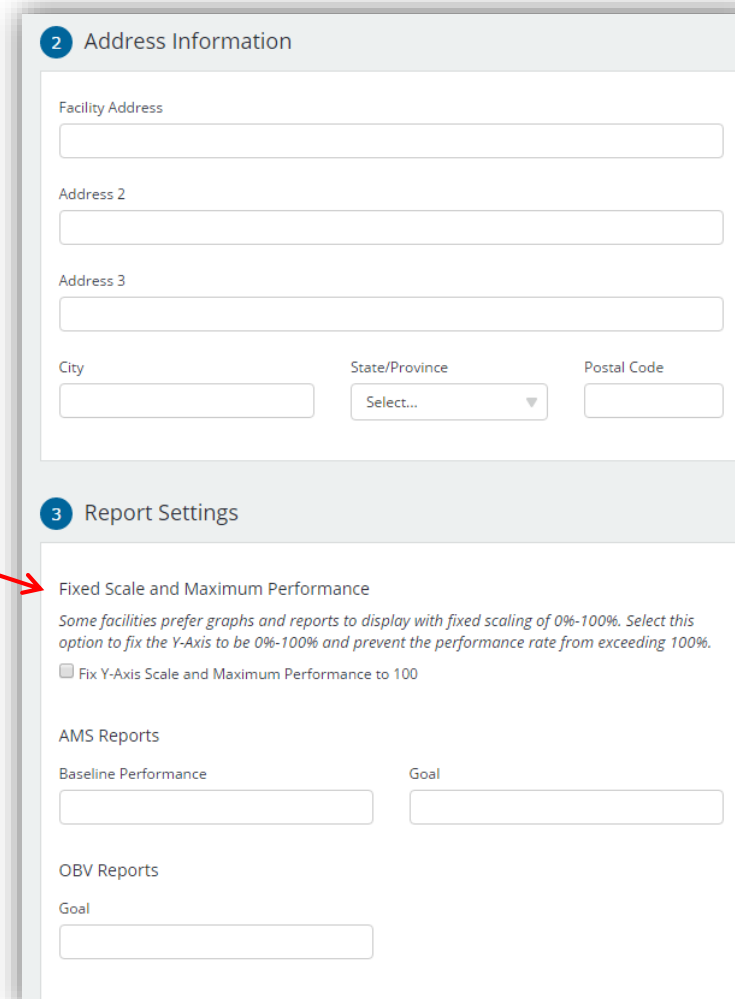
City State/Province Postal Code

3 Report Settings

Fixed Scale and Maximum Performance
Some facilities prefer graphs and reports to display with fixed scaling of 0%-100%. Select this option to fix the Y-Axis to be 0%-100% and prevent the performance rate from exceeding 100%.

Fix Y-Axis Scale and Maximum Performance to 100

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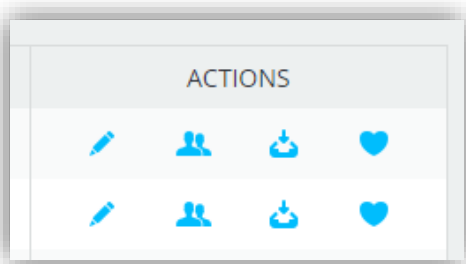
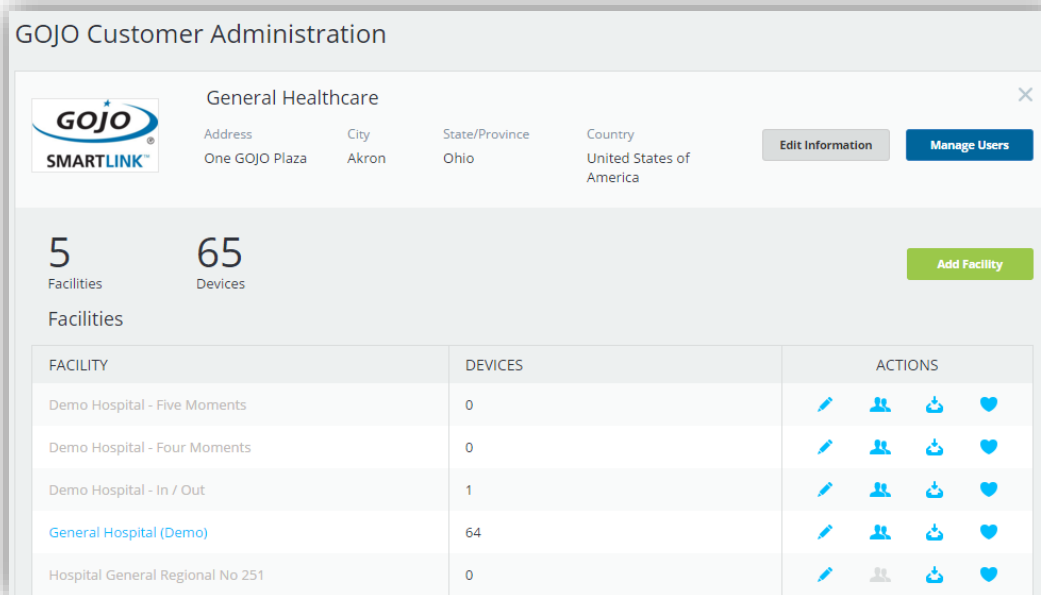
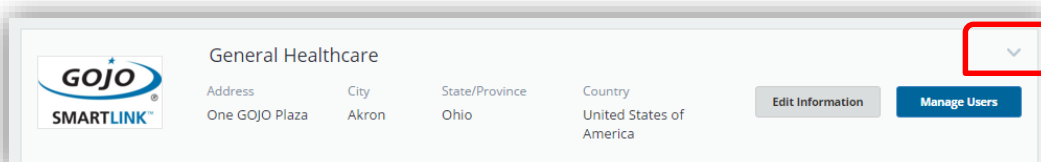
FACILITY	DEVICES
Demo Hospital - Five Moments	0

Edit Facility

1 Facility Basics

Facility Name
General Hospital (Demo)

Select Region
United States of America

Select Time Zone
(UTC-05:00) Eastern Time (US & Canada)

Select Location Template
US Hospitals

Select Hygiene Protocol
In and Out

Occupancy
100

Outbound Opportunities
Some facilities only need to record when someone leaves a room, or outbound opportunities. Select this option if you wish for your entire facility to operate and report this way.

Record Only Outbound Opportunities

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FACILITY	DEVICES	ACTIONS
Demo Hospital - Five Moments	0	   

Archiving Facility. Continue?

Cancel

Continue



Archived Facilities

FACILITY	DEVICES	RESTORE
Demo Hospital - Five Moments	0	

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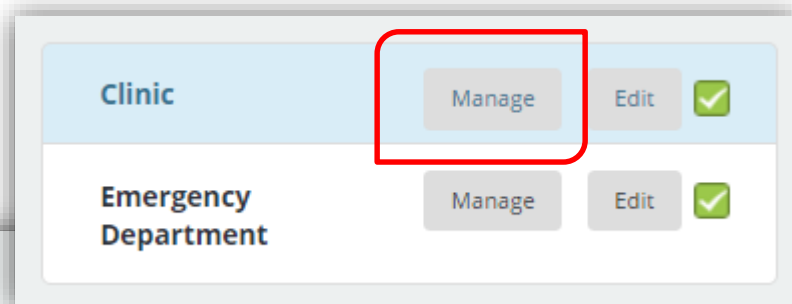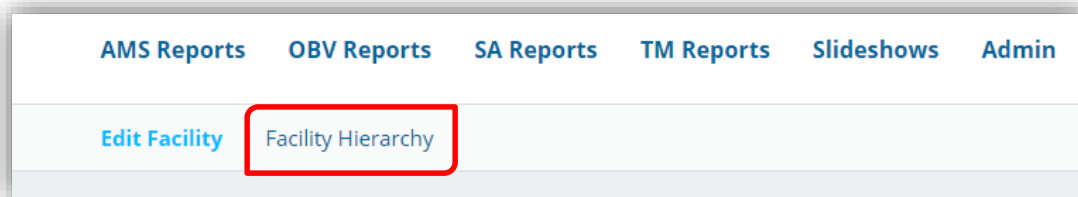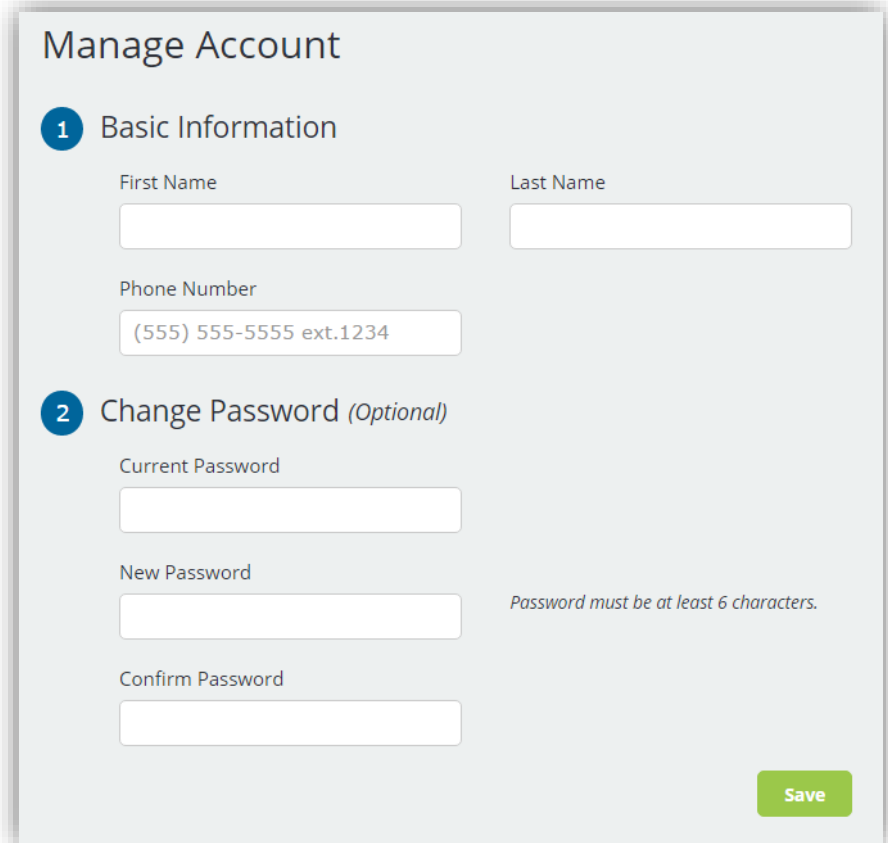
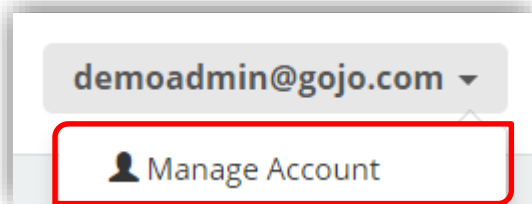
A screenshot of the 'Manage Account' form. The form is titled 'Manage Account' and is divided into two sections:

- 1 Basic Information**
 - First Name:
 - Last Name:
 - Phone Number:
- 2 Change Password (Optional)**
 - Current Password:
 - New Password: Password must be at least 6 characters.
 - Confirm Password:

A green 'Save' button is located at the bottom right of the form.

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